





RetroFit

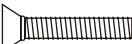


GynoCart

Latch Replacement

Hardware:

- 2x  10-32 Phillips Flat Head
- 8x  4-40 Hex Nut
- 2x  1" Threaded Step Washer
- 2x  3/4" Spacer

Tools Needed:

-  Phillips Screwdriver
-  1/4" Closed End
-  5/64" Allen Wrench

Latch Replacement:

NOTE: Take care not to drop hardware inside doors.

1. Remove and discard existing catches from cart body including four securing hex nuts and two spacer nuts from each catch assembly (see Figure A).
2. Remove and discard existing latches from cart doors.
3. Attach door latch cups using supplied 10-32 flat head screws and threaded step washers (see Figure B).

NOTE: Be sure that the hardware is secure before inserting the spacers and magnets. Once the magnet is in place, it can not be removed.

4. Attach central body catches using supplied 4-40 hex nuts (see Figure C).

